

INSTRUCTION FOR USE

Silicone Scar Sheets

Name: Silicone Scar Sheets

Type&Size

3cm×12cm	3.5cm×7.5cm	3.5cm×15cm
3cm×20cm	4cm×7.5cm	4cm×15cm
5cm×7.5cm	6cm×12cm	

Product Structure and Composition

It consists of polyurethane film, silicone gel and release paper.

1. Peeling strength: The 180 degree peeling strength of silicone scar sheets material should be $\geq 0.1\text{N/cm}$.
2. holding power: The slide should not exceed 2.5mm.
3. Ductility: Should be $\geq 0.25\text{N}/10\text{mm}$.
4. Breathability: water vapor transmittance should be $\geq 200\text{g} \cdot \text{m}^{-2} \cdot 24\text{h}^{-1}$.

Range of Application

It is used for scar care of new and old hyperplastic scars or neoplastic scars and newly healed wounds of intact skin.

Method of Use

1. Clean the scar with a neutral detergent and dry it before use.
2. Open the package and cut the sheet to suitable size for the wounds(For small scars, the sheets can be cut appropriately to completely cover the scar and slightly cover the skin around the scar. For larger scars, several sheets can be applied side by side to completely cover the scar.)
4. Remove backing paper.
5. Place the adhesive side of the sheet over the scar without stretching it. Although Silicone Scar Sheets are sticky, they can be fixed with an appropriate secondary dressing. Please be careful not to apply the dressing too tightly to the scar, otherwise it may cause discomfort to the scar and surrounding skin.
6. Each sheet can usually be used for 2 to 4 weeks, and can be replaced when the sheet is

not strong in viscosity. The recommended course of treatment is 4 to 6 months, depending on the scar condition.

7. Silicone Scar Sheets is suitable for 24 hours a day. You can remove the sheet to observe or wash the skin every day.

Contraindications

Do not use on open wounds.

Precautions

In case of skin infiltration or rash, stop using, wait for symptoms to subside, and gradually increase the treatment time. If symptoms persist, discontinue use and consult your doctor.

Storage

1. This product should be stored in a well-ventilated, clean and dry room.
2. This product should be carefully protected from heavy pressure, direct sunlight and rain and snow immersion during transportation.

Symbols On Labeling Recautions



Non-sterile



See Instructions For Use

Shelf Life : 3 years

Manufacture Date : Refer to inner box

Manufacturer

SLK Medical Material (Tianchang) Co., Ltd.
No. 33, Zhenglong Road, Qinlan Town
Industrial Park, Tianchang City, Anhui
Province 239300,P.R.China

Tel: 0512-67339692