

Silicone Foam Dressing

Moms, It Makes a Difference



Has the potential to prevent scarring and accelerate the healing process.



amigo care dhaka

Trusted Advance Wound Care Solution


Silicone Foam Dressing With Border

Size: 10cm x 20cm, 10cm x 25cm

Color: Pink, Skin

How Do Silicone Dressings Work?

Silicones are long-chain polymers that contain repeating chains of silicone, oxygen, and other elements. They have a hydrophobic soft silicone layer that is tacky to touch, preventing the dressing from sticking to the wound but still enabling it to adhere to the surrounding skin. This helps to minimise patient trauma during dressing changing.

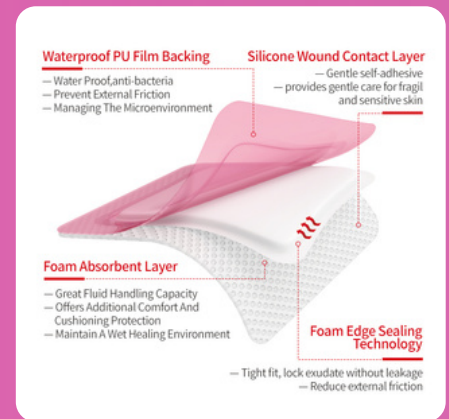

Help mothers heal with an advanced dressing

The initial days post C-section are crucial for the ongoing well-being of the mother. Clinicians play a pivotal role in ensuring a smooth recovery by carefully choosing the incision dressing. To balance standard care and enhance outcomes, medical teams should explore advanced dressings for C-section incisions. These specialized dressings not only shield the incision but also offer flexibility, absorption, easy removal, and technology aimed at preventing surgical site infections (SSIs).

5 Advantages for C-Section Incisions.

1. Flexibility - promotes early mobility without adding tension to the incision.
2. Protection - Minimizes healing interference & dressing related skin damage.
3. SSI Risk Reducing - creating a protective barrier, maintaining optimal moisture levels, and incorporating antimicrobial properties to inhibit bacterial growth.
4. Shower Proof - Allows for bathing and cleanliness.
5. Absorption - Ensure proper moisture balance necessary to healing.

Range of Application: It is used to cover non-chronic wound surface and absorb wound exudate.

Replacement Frequency: In the early stage of wound treatment, always check the condition of the foam dressing. the dressing can be applied to the wound for up to 7 days.

Contraindications:

1. Use with caution if the skin is allergic to silicone glue.
2. Do not use on third-degree burns, severe infection and tumor wounds.

We deliver prestigious certified products




amigo care dhaka
Trusted Advance Wound Care Solution

Silicone Foam Dressing With Border

Size: 10cm x 20cm, 10cm x 25cm

Color: Pink, Skin